evenflo®

**Advanced Manual
Breast Pump**

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*For video instructions about how to use and care for your pump,
visit: www.evenflofeeding.com/videos*

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Congratulations on the birth of your baby! Thank you for choosing Evenflo to support you on your breastfeeding journey.

For more information about breastfeeding or pumping, contact a lactation expert, your physician, or visit one of the websites listed below.

If you have questions about the performance or operation of this pump, please contact our Parent's Ally Resource Center.

USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at www.evenflofeeding.com.

Mexico: 01-800-890-32-40 or visit us online at www.evenflo.com.mx.

International Lactation Consultant Association

Lactation consultants with the IBCLC credential specialize in the clinical management of breastfeeding. There may be a fee to use lactation consultant services. To locate one near you, call: (919) 861-5577. Website: www.ilca.org

La Leche League

Provides free mother-to-mother breastfeeding support.

Phone: 1-800-LALECHE (US), or (847) 519-7730. Website: www.llli.org

Office on Women's Health

Provides trained breastfeeding peer counselors for free telephone help with common issues. Phone: 1-800-994-9662. Website: www.womenshealth.gov

WIC (Women, Infant and Children Program)

Provides breastfeeding help to mothers and children enrolled in the WIC program. Website: www.fns.usda.gov

Babies in the Workplace

Website: www.babiesatwork.org

Breastfeeding and Working

Website: www.workandpump.com

Kelly Mom Breastfeeding and Parenting

Information on multiple topics. Website: www.kellymom.com

Low Milk Supply Information and Support

For mothers experiencing low milk supply.

Website: www.lowmilksupply.org

Breastfeeding After a Reduction

Information for women who are breastfeeding after a breast reduction.

Website: www.bfar.org

Please Note: This pump is designed and intended for occasional use only, meaning **no more than 2 pumping sessions per day**. Refer to the chart on the back of these instructions for examples of appropriate situations for occasional use. If this pump is used more than twice daily, your pump's life expectancy may diminish earlier. Also, using this pump more than twice per day may impact your pumping success and/or your health. Moms who work full-time, or pump more than occasionally, should use the hospital strength Evenflo Advanced Double Electric Breast Pump, which has been expertly developed for frequent pumping.

Indications for Use: The Evenflo Advanced Manual Breast Pump is intended to express and collect milk from a lactating woman's breasts. This pump is intended for a single user.

Once the package has been opened, this product may not be returned to the store. If you have any issues or concerns with the product, please contact our Parent's Ally Resource Center.

Table of Contents

Important Safeguards and Warnings	2
Identification of Parts	2
Before First Use	3
Assemble Your Breast Pump	4
Check the Fit of the Flange	6
Using Your Breast Pump	7
After Every Use	8
Storing Your Breast Milk	8
Using Your Breast Milk	9
Troubleshooting	10
Limited Warranty	11


▲ IMPORTANT SAFEGUARDS AND WARNINGS

Read all instructions before using and save for future use.

- Use this product only for its intended use as described in these instructions.
- To avoid damage, **DO NOT** sanitize parts in the microwave after initial use.
- **NEVER** sanitize handle in the microwave.
- Use pump gently. Hard, rapid pumping can cause nipple irritation or injury.
- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds. See page 9 for proper use of breast milk.
- **DO NOT** re-freeze breast milk. After feeding, any unused breast milk left in the bottle can be refrigerated and used within 4 to 6 hours. See page 8 for proper milk storage.
- **DO NOT** use this product if it is broken or any parts are missing.
- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening. See page 6 for proper flange fit.
- If you are experiencing engorgement, prevent possible injury by discontinuing use of this pump and consulting a medical provider.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- Breast pumps are personal care items, for one user only.
- **ALWAYS** test temperature of liquid before feeding.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.


• Save These Instructions for Future Use.

Identification of Parts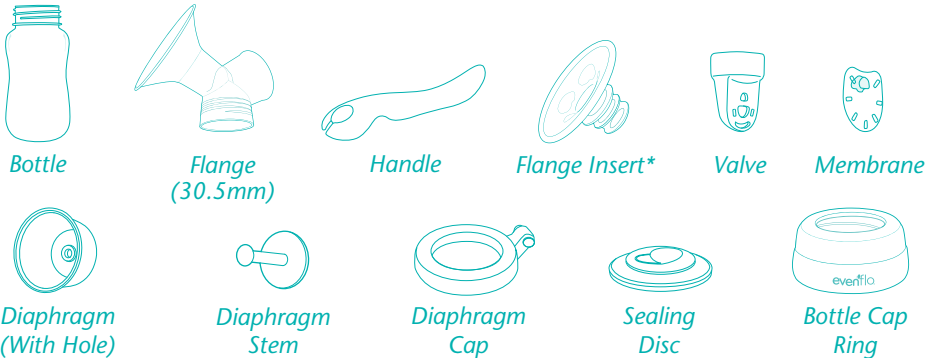


* Additional sizes are available separately. See page 6.


IMPORTANT: Before using your breast pump, disassemble and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.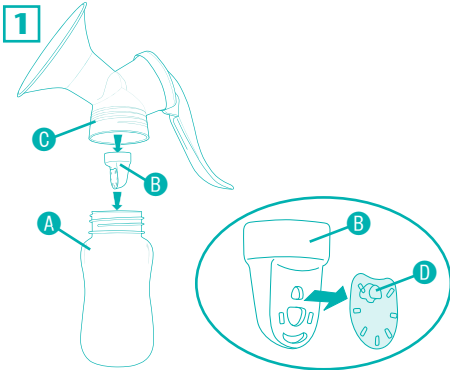


Unscrew bottle **A** and twist valve **B** to remove from assembled flange unit **C**. Remove the membrane **D** from the valve **B**.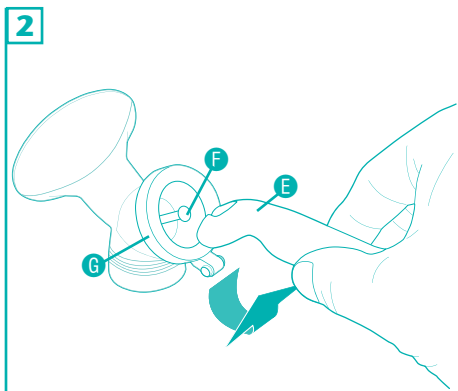

Grasp the middle of the handle **E** and lift in an upward motion to disconnect from the diaphragm stem **F** and diaphragm cap **G**.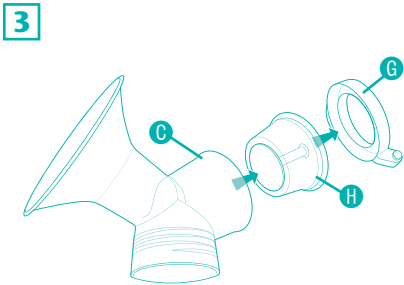

Remove diaphragm cap **G** and diaphragm assembly **H** from flange **C**.

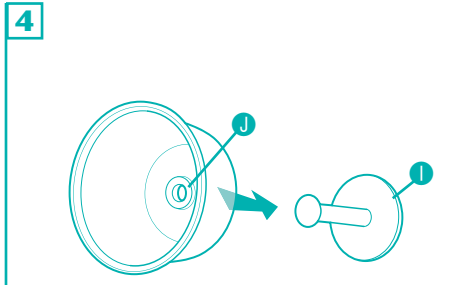Pull diaphragm stem **I** through the hole in the base of the diaphragm **J**.

Before First Use

To Sanitize Parts on the Stove:

Place disassembled pump parts in a pan of boiling water for 5 minutes. **Stir frequently. DO NOT** allow parts to rest on bottom of pan as they may melt or become damaged.

To Sanitize Parts in the Microwave:

All pump parts, **except handle**, may be used with a microwave sanitization device. Please follow device manufacturer's instructions.

Handle

▲ IMPORTANT


DO NOT CLEAN HANDLE IN MICROWAVE or with sanitization bags as it will damage the handle and the PUMP WILL NO LONGER WORK.

Before EVERY Use

- Wash your hands with soap and water before handling your pump.
- Ensure parts are clean and dry. Refer to cleaning instructions on page 8.
- Use care when handling the valve/membrane assembly to avoid damage.
- Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.

Assemble Your Breast Pump

1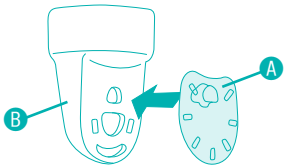


Attach the membrane **A** to the valve **B**.

Note: Make sure the membrane is attached and is lying flat against the valve, or the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this also will cause decreased suction and the pump to not work properly.


Assemble Your Breast Pump

2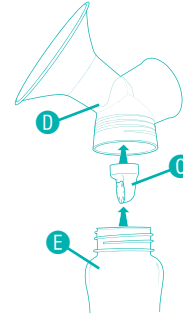


Push valve assembly **C** onto the bottom of the flange **D**. Screw bottle **E** into bottom of flange.

3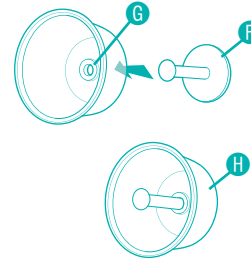


Push diaphragm stem **F** through the hole in base of the diaphragm **G**. Pull stem upward until the base of the stem lays flat against diaphragm to create diaphragm assembly **H**.

4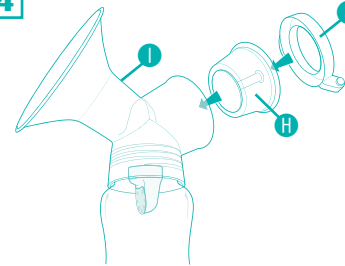


Place diaphragm assembly **H** in flange assembly **I** and snap diaphragm cap **J** onto flange.

5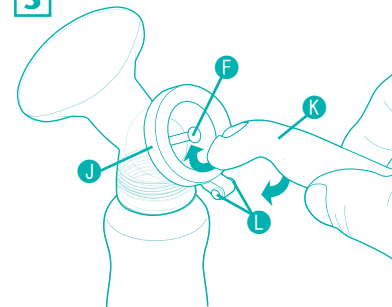

Insert handle **K** in a scooping motion around middle of stem **F**. Align handle with pin **L** on diaphragm cap **J** and push downward to snap into place.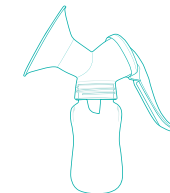

Note: Handle pivots for pumping comfort. (See p 7)

Check the Fit of the Flange


The Advanced Manual Pump works with the Evenflo AdvancedFit™ Flange System. The Advanced Manual Pump comes with the Large 30.5mm base flange size and a silicone flange insert. Additional sizes are available separately for purchase, allowing you to find your best fit.

The AdvancedFit™ Flange System available sizes include:

- X-Small = 21.0mm (insert)
- Small = 24.5mm (insert)
- Medium = 28.0mm (insert)
- Large = 30.5mm (no insert)
- X-Large = 33.0mm (insert)
- XX-Large = 36.0mm (no insert)

Correct Flange Size Vs. Too Small or Too Large


A proper flange size is needed for comfort and to maintain your milk supply. As your flange size can change several times throughout your journey, ensure your pump has multiple sizes available.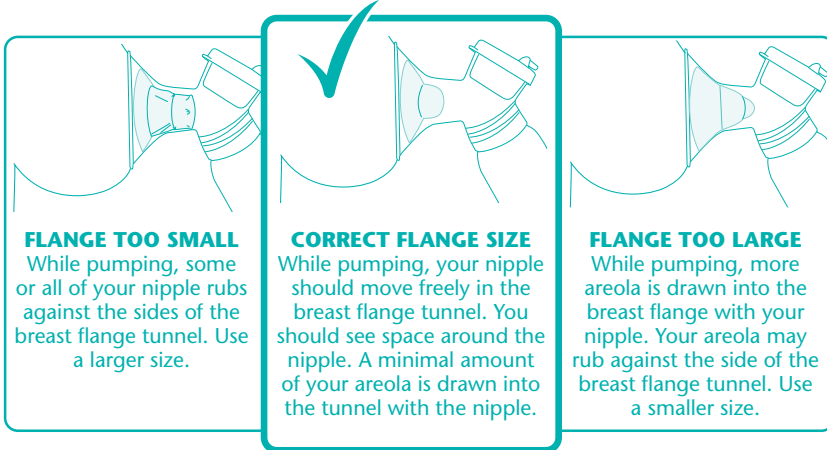

IMPORTANT:


- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort.
- **Included flange sizes may not fit all breast and nipple sizes. Additional sizes can be purchased separately.** If you are having issues after trying all included sizes, please contact our Parent's Ally Resource Center for assistance.

Using Your Breast Pump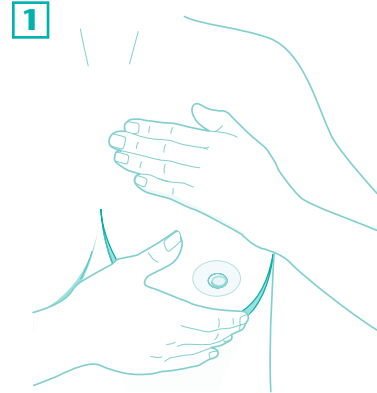

- To maximize your pumping results, stimulate your let-down reflex by:**
- Relaxing and thinking about your baby.
 - Gently massaging your breasts, starting nearest your ribs and working down toward the nipple.
 - Applying warm, moist compresses. Continue until you can hand-express a few drops of milk.

Now you should be ready to use your pump.

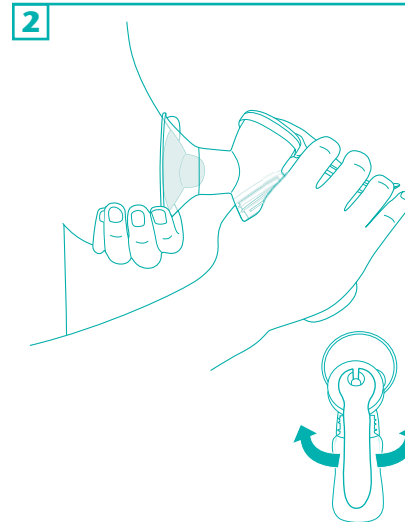- Place the breast flange to you breast centered over the nipple.
- Push the flange slightly towards your breast and allow the breast to completely fill the flange (this will ensure a good seal).
- Begin squeezing and releasing the handle in a gentle, rhythmic motion. You will notice that squeezing the handle draws the milk; releasing the handle allows the milk to collect in the bottle.
- The pivoting handle allows you to adjust the position at which you are pumping to maximize your overall comfort.

Note: Supporting your breast with your free hand and compressing slightly while pumping can help your milk flow easier.

IMPORTANT! Avoid hard, rapid pumping, which can cause nipple irritation or injury.

- Continue pumping until milk is no longer flowing or bottle is full.
- Massage your other breast to stimulate a let-down, as before, and repeat the steps above. If desired, massage the first breast again and repeat on both sides to maximize expression.

3

- When pumping is complete, release pump handle and remove pump from the breast.
- Carefully unscrew bottle from pump. Place the sealing disc into the cap ring and screw securely onto the bottle. Your breast milk is now ready for storage.

IMPORTANT! If you feel pain in your breast or nipple, stop pumping and slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort while pumping, check flange fit to ensure you have the most appropriate size (p. 6) and contact your healthcare provider for assistance.

After Every Use

To ensure that your breast pump performs well, follow these steps after every use:

- Completely disassemble the flange assembly, see page 3.
- Rinse the flange assembly in cool water to remove milk residue.
- Hand wash with warm, soapy water.
- Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
- Use care when handling the valve/membrane assembly during cleaning to avoid damage. Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.
- Go online or contact Parent’s Ally to find replacement parts.

Storing Your Breast Milk

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart below as a guide for storing your breast milk for future use.

Note: In hot weather, chill milk immediately to prevent spoilage.

Breast Milk Storage Guidelines*				
	Room Temperature	Refrigerator	Self-contained Refrigerator Freezer	Deep Freezer
Freshly Expressed Breastmilk	4 hours at 73 – 77°F (23 – 25°C)	5 – 7 days at 32 – 39°F (0 – 4°C)	3 – 4 mos. at 0°F (-19°C)	6 – 12 mos. at 0°F (-19°C)
Thawed Breastmilk (previously frozen)	DO NOT store	24 hours	NEVER re-freeze thawed milk	NEVER re-freeze thawed milk

Note: Breast milk should be stored in the main refrigerator or freezer compartment, away from the door, to avoid changes in temperature.

Using Your Breast Milk

1. Thaw slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, warm the milk by:
 - Using a bottle warmer
 - Holding the sealed bottle under warm, running water, or
 - Heating water in a cup or other small container, then placing frozen milk in the water to warm.
3. The milk fat may separate during thawing. If so, gently swirl milk to mix before offering it to baby.
4. If feeding from the collection bottle, **you must use Evenflo Balance + Standard Nipples**. Other brands or types of nipples will not work effectively. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the **TOP** of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care to not over or under tighten. *Evenflo Balance + Standard Nipples sold separately.*
5. Always test the temperature of the milk before feeding.

IMPORTANT!

- **NEVER microwave human milk or heat it directly on the stove.**
- Previously frozen milk may be kept in the refrigerator for up to 24 hours after it has finished thawing. **DO NOT re-freeze.**
- **If baby does not finish milk at one feeding**, it may be refrigerated and offered at the next feeding before it is discarded (4 – 6 hours).
- **DO NOT pour freshly pumped breast milk onto frozen breast milk** because thawing can occur.

* Human Milk Banking Association of North America. Best practice for expressing, storing, and handling human milk in hospitals, homes, and child care settings. Raleigh, NC: 3rd Edition, 2011.

Problem: LOW OR NO SUCTION	
Possible Cause	Action
Valve not assembled to pump unit.	Mount the valve firmly to pump unit (p. 5).
Membrane torn or not assembled to pump unit.	Replace membrane or ensure that it is connected properly to the valve.
Membrane not sealing.	Make sure valve and membrane are clean with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve (p. 4).
Dried milk in pump pieces.	Follow cleaning instructions. This pump can be easily disassembled and washed (p. 8).
Not a good seal on your breast.	Reposition the flange to get a tighter seal. If the breast becomes wet with milk, dry skin and flange and start again (p. 7).
Problem: DIFFICULT MOVEMENT OF PUMP HANDLE OR HANDLE WILL NOT RETURN	
Possible Cause	Action
Dried milk or debris in pump or handle improperly installed.	Clean pump handle or review installation instructions (p. 4).
Problem: MILK WON'T FLOW INTO THE BOTTLE	
Possible Cause	Action
The valve is not opening.	Clean the valve thoroughly and be sure it opens (p. 8).
No let-down.	Attempt to stimulate the let-down reflex (p. 7).

Still having problems? Call our Parent's Ally Resource Center for help.

Remember, this is a personal care item for one user only. For hygienic reasons, it **CANNOT BE RETURNED TO THE STORE ONCE PACKAGE HAS BEEN OPENED.**

If you have any questions, comments, or concerns about our Evenflo breast pumps, please contact our Parent's Ally Resource Center.

This product is designed for use by one person. Use of this product by more than one person will void the warranty.

For a period of 90 days from the original purchase of this Product, Evenflo Feeding, Inc. ("Evenflo") warrants to the original end user ("Purchaser") this Product (including any accessories) against defects in material or workmanship. Evenflo's sole obligation under this express limited warranty shall be, at Evenflo's option, to repair or replace any Product that is determined to be defective by Evenflo and determined to be covered by this warranty.

Repair or replacement as provided under this warranty is the exclusive remedy of the Purchaser. Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service. This express limited warranty is extended by Evenflo ONLY to the original Purchaser of the Product and is not assignable or transferable to subsequent purchasers or end users of the Product. For warranty service, contact Evenflo's Parent's Ally Resource Center at 1-855-EFI-BABY (334-2229) or www.evenflofeeding.com.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THIS PRODUCT, OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Parent's Ally

- USA/Canada: 1-855-EFI-BABY (334-2229); www.evenflofeeding.com
- Mexico: 01-800-890-32-40; www.evenflo.com.mx

Manufactured and Distributed by Evenflo Feeding, Inc.

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Fabricado y distribuido por Evenflo Feeding, Inc.